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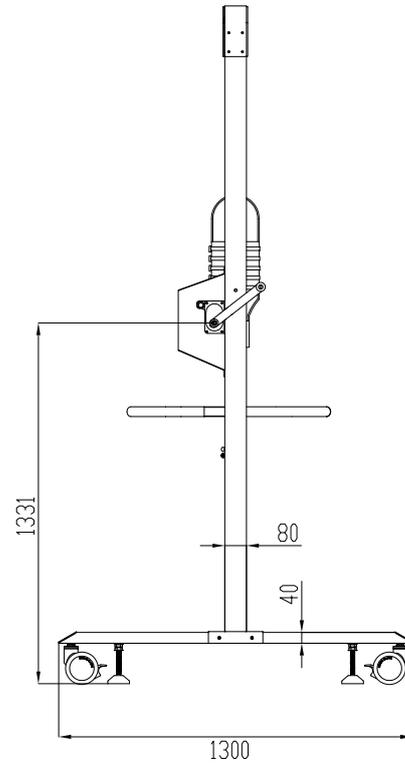
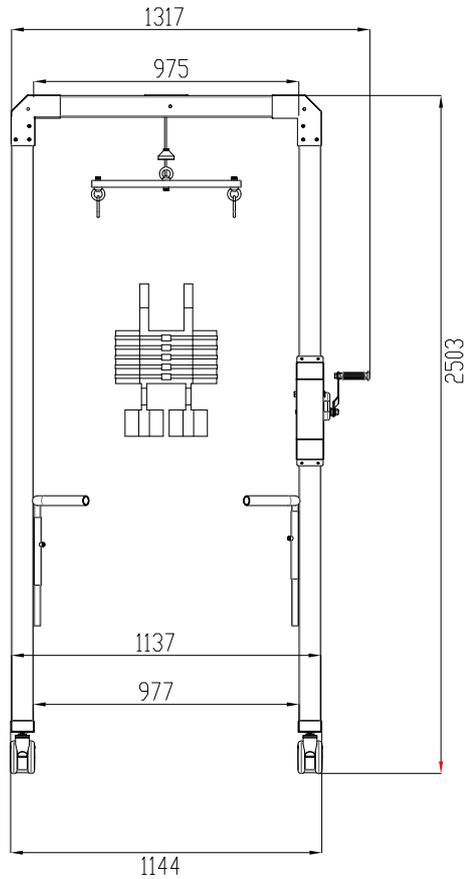
INTRODUCTION: This Safe walker works through control unit to lift user up or decreasing body loading of waist a & lower extremity. So that user can stand, walk and gait training safely, and gradually increase the ability of user /s lower limbs to support their own weight by walking, and aim at restore walking function.

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2、FEATURE AND USAGE

(1)weight reducing height range : 0~600mm, tolerance ± 10 mm.

(2)Machine dimensions (L×W×H) : 1317×1300×2053mm , Tolerance ± 10 mm

3.PRODUCT STRUCTURE AND ASSEMBLY

This product consists of hanging frame, posts, hanging sling, handrail, protection vest, base frame, transport wheels and so on composition.

4 .APPLICATION

Suitable for patients with walking disabilities in the supporting training.

5.CONTRAINDICATIONS

Acute infectious diseases, Severe heart disease, Severe hypertension, Severe anemia, Arteriosclerosis, Pregnancy and menstruation in women, Skin allergies, Skin damage, Pulmonary dysfunction, Severe diabetes, Osteoporosis Heavy training condition).

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- (1).Be sure to carefully read this manual before use, and follow the requests noted to operate it.
- (2).It is recommended to use this product under the guidance of a doctor.
- (3).The children, the elderly, emergency patients and patients without self- care ability should be used under the -supervision of a doctor .
- (4).Inspect the hanging sling and suspension rope before and after usage, whether there are open lines, rupture, safety buckle failure and other conditions , it should be promptly replaced if found any.
- (5).Follow training rule of II gradual and orderly II , increase the lower limb load and training time gradually.
- (6).Block Hanging sling by insert the buckle into socket is a must.
- (7).Carefully inspect the suspension rope and hanging sling, avoid it loosen lead to harmful to the user.
- (8).Weight less gait training should be operated on a solid ground.

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(9). Explanations of the graphics, symbols, abbreviations and other contents used in the label.

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

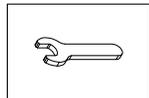
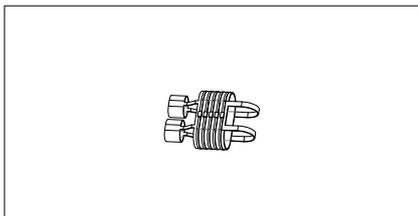
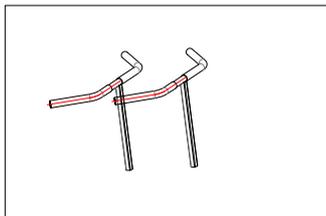
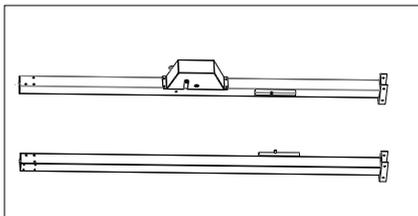
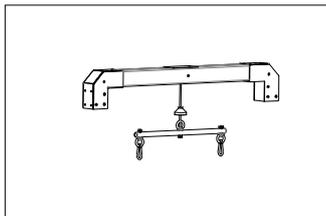
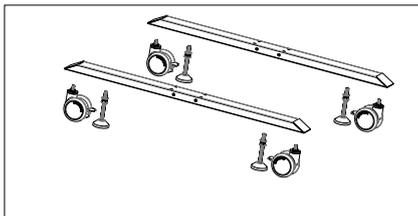
Step1: Open the packaging and remove the left and right foot pad tubes, left and right frame and other parts, Place as shown.(Use cardboard to isolate the ground contact area to prevent damage or scratches.)

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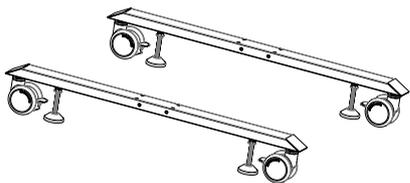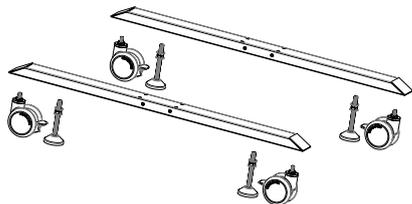
Step2: Take out the left and right foot pad group, universal foot wheel and mechanical adjustment seat. As shown in the figure, fix the universal foot wheel and mechanical adjustment seat on the left and right foot pad tube group, and lock it with tools;

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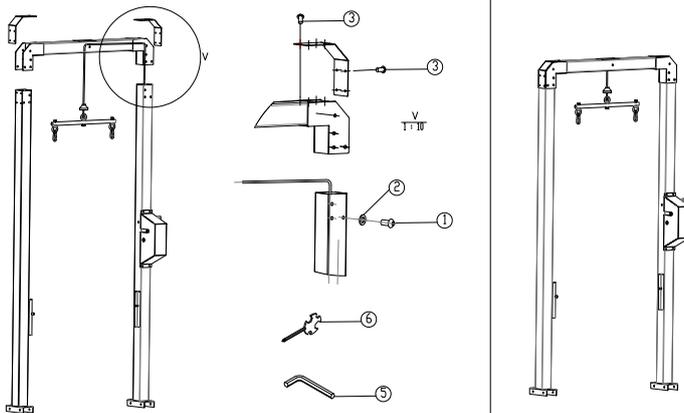
Step3: Remove the left frame, right frame, top frame, and side cover. First, detach the screws and washers from the left and right frames and top frame, then secure them as shown in the diagram. Reattach the removed components using the original screws and washers, and tighten them with tools. (Note: The right frame and crossbeam have tie-down ropes on the right side. Ensure the ropes remain attached during installation to prevent the pulley grooves from slipping.)

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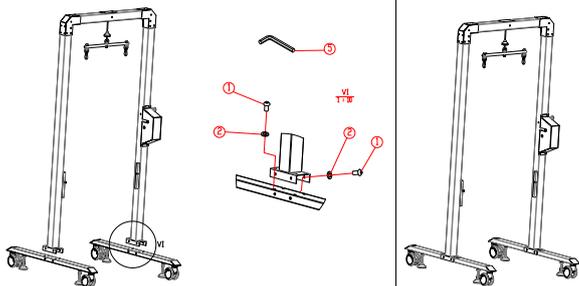
Step4: First, remove the screws and washers from the left and right foot pad tube group and put them aside. Then, fix the assembled left and right frame on the left and right foot pad tube group as shown in the figure. Finally, connect and fix them with the removed screws and washers, and lock them with tools;

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
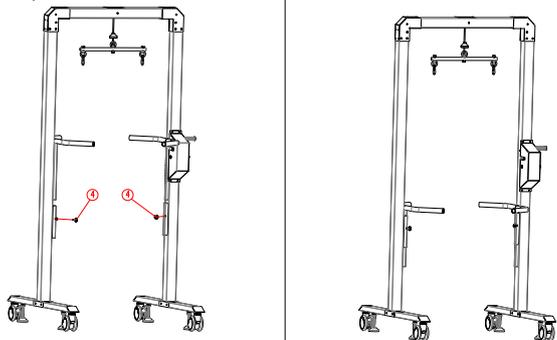
Step5: Push the assembled machine forward with at least four people working in coordination to prevent tipping over. Insert the left and right handle bar into the left and right frame, then secure them with adjusting knob to maintain consistent height on both sides. (Adjust the height according to the user's height during use and ensure tight locking.)

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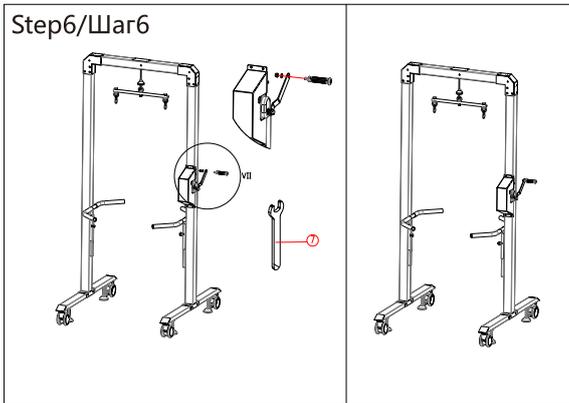Step6:First, remove the flat washer and nylon cap from the handle and set them aside. Then, take out the handle and fix it on the sheet metal of the winch in the position shown in the picture. Use the removed flat washer and nylon cap to connect and fix it, and finally lock it.

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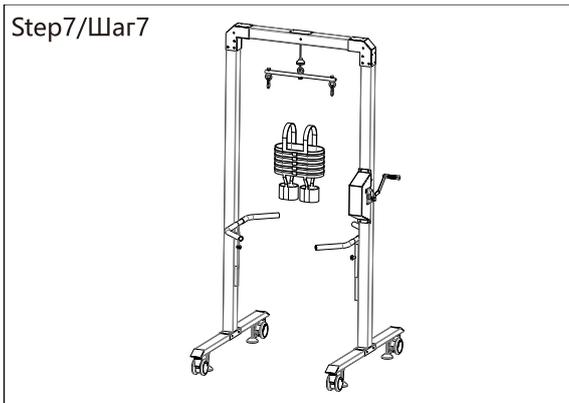Step7:The user wears the air bag and is fixed on the suspension rod group. When the height is adjusted to a comfortable level, it can be

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8. OPERATION

Operation steps

- (1). Move user sitting on wheel chair to Safe walker, wear and adjust protection vest in comfort.
- (2). Adjust handrail in height fit, trainer helps user stand up and hook up user vest to the suspending set and adjust based on user's height.
- (3). Rotate handle to lift up user gradually till user feels weight reduced affordable. Start walking training.

9. MAINTENANCE

- (1). The parts that come in contact with the user shall be disinfected with a swab of medicinal alcohol at the end of each time training.
- (2). Keep the device clean and pay attention to moisture, dust. If long time no use, cover the protective cloth. Please note that do not place heavy objects on the device.

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(3) .Every 2 years , It is recommended for regular repair and maintenance to ensure stable performance of the device and continue to use.

(4) .Connect with dealer for repair if any out of order not to tear off part a t own.

10. TRANSPORTATION

Put poly foam between instrument and the outer packing carton to prevent the shock , the violent vibration and the humidity during the transportation .

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The background features several decorative hexagonal shapes. On the left, there is a vertical stack of three hexagons: a light gray one at the top, a white one in the middle, and a white one at the bottom. On the right side, there are several gray hexagons of varying sizes and orientations, some overlapping each other. At the bottom left, there are three solid-colored hexagons: a large gray one, a medium black one, and a small black one.

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